

3. ASSEMBLY OF YOUR i-PRESSURE™ SENSOR

3.1 : Install BATTERIES (AA) as specified in section 8.1. When putting batteries in, follow +/- indications found inside the compartment. Ensure that the red EXTRACTOR cloth is positioned for an easy removal of the batteries, as shown in image below. Re-install the batteries and close the compartment.



Extractor



1st battery



2nd battery

3.2: OPEN the cushion inflator valve in full by turning counter-clockwise.



Open valve

3.3: OPEN adaptor valve by partially UNSCREWING nut set. CONNECT the sensor to the cushion by inserting the cushion valve into the 90° adaptor. Hold together and TIGHTEN nut set by turning clockwise. Make sure the adaptor is placed parallel to the cushion.



Loosen



Connect



Tighten

3.4: Adaptator Assembly

- Connect the adaptator parts in accordance with the order illustrated below.
- Make sure to properly connect all parts the right way by referring to the illustration below.
- Note that you can easily tell Part #4 apart from Part #5 by the fact that Part #4 is white.



- 3.5:** ATTACH unit to the chair's frame using strips found inside the package and using handles located on the side of the unit. Strip may be split in half for dual fixation points.



ONE VELCRO® strip



TWO VELCRO® strips

- 3.6:** The i-PRESSURE™ sensor should always be kept easily accessible to the user and the caregiver to facilitate audible and/or visual monitoring of the device. An access to open and close the inflator valve is necessary at all times. The device and/or tubing should not be left hanging, tangled, pinched, or become potentially harmful by its location.

⚠ WARNING! Do not use device as a handle to move your cushion.

⚠ WARNING! Make sure the tubes are not hanging down, tangled, pinched or twisted. Choose a safe location to place the device.