

## 4. OPERATION AND SETUP

Now that you have completed the assembly of your i-PRESSURE™, you are ready to setup and operate your new sensor.

**⚠ WARNING!** The initial installation and calibration must be done by your healthcare professional.

### 4.1: Turning unit ON/OFF

Press/release target button to turn ON unit

- An initialization sound will be heard
- LEDs will flash one after the other

Press/release button (target) to turn OFF unit

- A closing sound will be heard
- LEDs will flash one after the other



**NOTE:** The device is AUTOMATICALLY TURNED ON when the user sits on its cushion.

**4.2:** INFLATE, POSITION user and ADJUST cushion according to the cushion manufacturer's recommendations. Use the cushion pump on the inflator valve found on the i-PRESSURE™.

**NOTE:** The cushion must be inflated AFTER the adaptor valve is installed on it.

### 4.3: RECORD your custom user/therapist setting

Once your cushion has been properly inflated and adjusted by your healthcare professional, it is time to record your custom cushion setting. Follow the 4 easy steps below to record your individual calibration.

Turn unit ON

Press and hold target button until 1 beep is heard, release button

Press and hold target button until 2 consecutive beeps are heard, release button

- LEDs will slowly flash sequentially: RED, GREEN, RED

To record pressure: press target button until a confirmation sound is heard

**NOTE:** The custom setting must be recorded while the user is properly positioned on its cushion.



**WARNING!** Unit will automatically exit calibration mode after 60 seconds. An error sound will be heard. This signals that the custom calibration was not recorded and remains unchanged.



**WARNING!** Make sure the individual recorded setting is appropriate.

---